


Abdominal compression bandage for horses

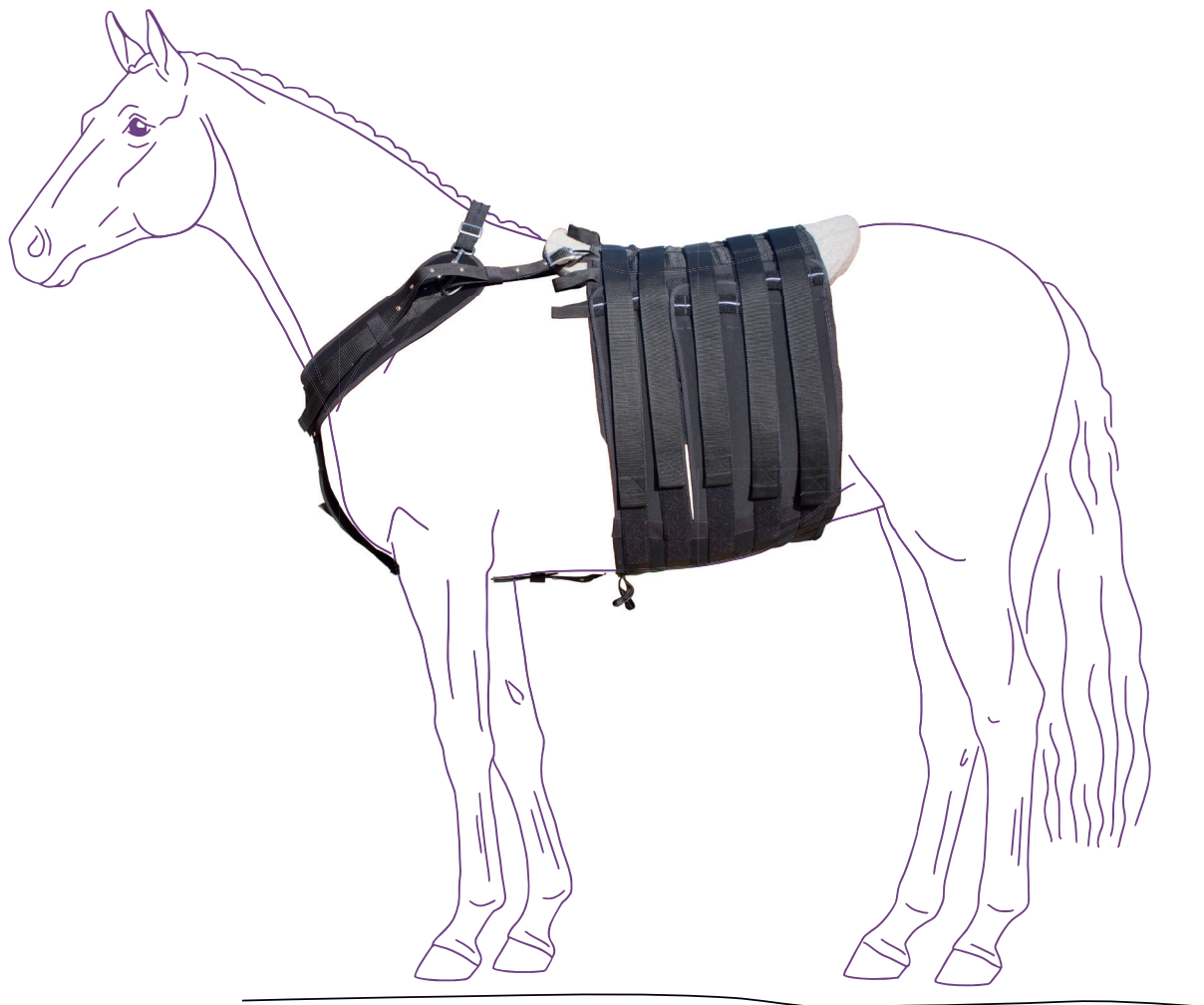

The alternative to abdominal dressing – the abdominal compression bandage from vetal

Can be put on by one person in just a few simple steps – quick and highly adaptable

The abdominal compression bandage replaces the abdominal dressing after colic surgery.

- ◆ It provides protection against wound infection and overloading of the seam
- ◆ minimizes the risk of scar perforation
- ◆ and counteracts the formation of lower abdominal edema.

After instruction, the use is also possible by laypersons. This allows for longer treatments for improved healing, even beyond the clinic stay.



stabilized
wound area

Solid canvas provides stability in combination with partially elastic bandage material, which ensures optimal adaptability to the horse's belly shape.

The sturdy Velcro straps allow for continuously adjustable compression.

The correctly applied abdominal compression bandage fits seamlessly to the body, thereby preventing common urination problems (stallions and geldings).




Abdominal compression bandage

- ◆ Washable at 40 degrees
- ◆ Reusable
- ◆ Quick and easy to apply, even by a single person
- ◆ Facilitates daily monitoring, cleaning and care of the wound
- ◆ No need for daily dressing changes.
All you need are wound dressings; no other dressing materials are required



Wound dressing pocket

- ◆ Wound dressings are placed in the replaceable pocket, which is secured with a Velcro fastener.
- ◆ The wound dressing pocket is made from 100% cotton and is machine washable at 95°C.
- ◆ It can be reordered either individually or in packs of 10.

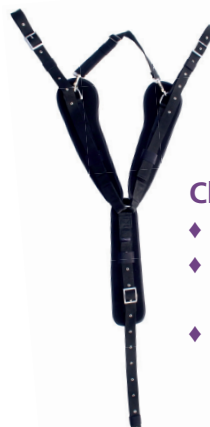

Chest harness

- ◆ individually adjustable
- ◆ prevents slipping of the bandage backward
- ◆ its elastic parts maintain the freedom of movement




Without padding of the back, the application of abdominal compression bandage can cause pressure sores or chafing – we therefore recommend the saddle pad to protect the horse's back.

The specially designed, high-quality wool felt saddle pad compensates the pressure through optimal padding and ensures the best possible skin climate. 100% pure wool felt is a renewable raw material, has moisture-regulating properties, is insulating, skin-friendly, breathable, pressure-elastic, and durable.

For care, we recommend vacuuming, brushing and, if necessary, hand washing with a little mild shampoo.

In case of a hollow back, a saddle pad with a free spinal canal can be made upon direct request.

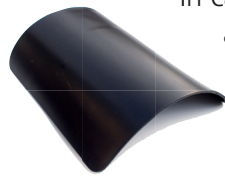

In case of the appropriate veterinary indication a compression plate for additional pressure from below on the wound area is available. The compression plate is placed below the wound dressing pocket and inserted into the abdominal compression bandage.



manufacturing in Germany -
competent contacts –
quickly available




vetal | | Orthopedic technology for animals by albrecht GmbH
Chiemseestraße 81 | 83233 Bernau am Chiemsee | Germany
phone. +49 8051 96129-0 | fax +49 8051 96129-30 | info@vet-al.de

www.vet-al.de

Measurements & Order

Purchaser
Customer number
Street and house number
Postal code and city
phone
e-mail

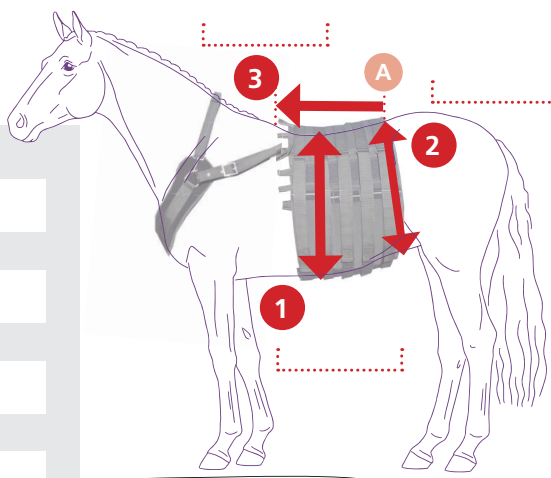

Back-measurement cm	Abdominal circumference cm	Size
30	115 – 150	XS
40	140 – 180	S
40	160 – 200	S long
45	180 – 220	M
45	200 – 240	M long
50	220 – 250	L
55	250 – 280	XL

Take these 3 measurements

- 1 measure the abdominal circumference at the widest point
- 2 Measure the circumference at the narrowest point in front of the hindquarters. Point A, where the tape measure rests on the top of the back, is the starting point for the next measurement
- 3 from there, measure the length to the withers (= back measurement)

Article name



Abdominal compression bandage

For measurements/sizes, see chart. If the abdominal circumference is close to the largest measurement in a size, choose the next size up.



Standard saddle pad

Delivery will match the abdominal compression bandage ordered. If you require a different size, please specify.

Saddle pad with free spinal canal on request.

Chest harness




The chest harness is adjustable and available in two sizes. We supply the smaller chest harness (size M) for the abdominal compression bandages up to size M. Starting from size M long, you will receive the larger chest harness (size L). If you require a different size, please specify.

Wound dressing pocket

Fixation via Velcro fastening.



Pack of 10 wound dressing pockets

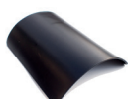

Compression plate

Optional, increases compression on the wound area. We supply a compression plate matching the size of the abdominal compression bandage you ordered.

Quantity Size Article number*

* see separate article number overview